• Non indurente • pH > 12 • radiopaco

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CALCIUM HYDROXIDE PREPARATION

8 NEEDLES

Contents:
4 x 1.2 ml syringes CONTROL
8 pre-bent needles

La Maison Dentaire SA is a subsidiary of Medirel SA

FOR PROFESSIONAL USE BY DENTISTS

Rev. 11/2021

EC REP / Importer
La Maison Dentaire SA
Alte Churerstrasse 36
FL-9496 Balzers - Liechtenstein

MEDIREL SA
via Redondello 20, CH - 6982 Agno
Tel. 0041 (0) 91 604 54 34
Fax 0041 (0) 91 605 30 26
info@medirel.ch www.medirel.com

• Non-setting • pH > 12 • Radiopaque

PRODUCT DESCRIPTION

CONTROL is a smooth, creamy, non-setting calcium hydroxide preparation in an aqueous hydroxyethylcellulose base with barium sulfate. It is recommended for all clinical applications where calcium hydroxide paste is indicated. It is a dressing, not a hard base.

INDICATIONS

Direct Pulp Capping, Pulpal Curettage, Pulpotomy, Dentin Bridge Formation, Temporary and Intermediate Root Canal Treatment, Periapical Cysts, Abscesses and Lesions, Root Resorption, Root Perforations, and Apexification.

NOT for use as a hard base.

VITAL PULP TREATMENT

CONTROL is applied directly on the pulp in all pulp capping, pulpotomy and pulpal curettage procedures. It stimulates the pulp to form a new dentin bridge which is visible clinically after 2 - 3 months.

DIRECTIONS FOR DIRECT PULP CAPPING:

1. Place a 22-gauge applicator tip on the CONTROL syringe and apply a very thin film of CONTROL on all exposed dentin and dry with a slow flow of air.
2. Place a bead of CONTROL 3mm thick directly over the exposed pulp and dry with a slow flow of air until a crust appears. CONTROL does not set hard.
3. Gently tamp the paste with a pellet of cotton to make sure it is in contact with the pulp.
4. Place a thin mix of hard setting cement over CONTROL.
5. After cement hardens, place final restoration.

Note: For vital pulpotomy and pulpal curettage procedures, pack pulp gently with sterile cotton until blood clots, then proceed as above. A local anesthetic may be used.

ROOT CANAL TREATMENT

Use routinely as an intracanal dressing between office visits and as a dressing for cysts, abscesses, lesions, etc. Also use for apexification and other hard tissue formation.

Directions for Root Canal Therapy:

1. Prepare and cleanse canal in conventional manner.
2. Place a 22-gauge applicator tip on the CONTROL syringe and fill canal with CONTROL.
3. Seal access opening with your preferred cement.
4. CONTROL is easily removed with file and irrigation.
5. In cases requiring extended root canal therapy, change dressing every month.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature.

Avoid direct light, extremes of temperature, contamination and sources of ignition.

- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

Note: Use a new applicator tip for each patient. To extend shelf life of CONTROL, place a drop of water in the cap and recap syringe promptly after each use.

CAUTION

May be irritating to eyes, skin, respiratory system, gastrointestinal tract or mucous membranes, especially in large quantities and/or with prolonged exposure. For use by dental professionals only. Wear gloves, safety glasses; wash hands after use. Avoid unnecessary exposure.

REPORTING OF SERIOUS INCIDENTS

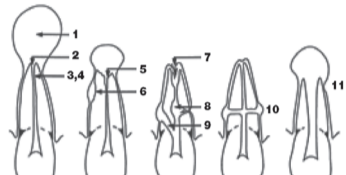
Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Medirel SA
Via Redondello 20, CH-6982 Agno / Switzerland
Tel: +41 (0)91 604 54 34 / Fax: +41 (0)91 605 30 26
email: info@medirel.ch

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient.

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

CLINICAL SITUATIONS WHERE CALCIUM HYDROXIDE CAN BE USED

1. exudation control
2. large periapical lesions
3. antibacterial dressing
4. temporary root filling
5. apical inflammatory resorption
6. inflammatory resorption following trauma
7. apical internal resorption
8. internal-external root resorption
9. perforations
10. transverse root fractures
11. incompletely developed pulpless teeth (Heithersay, G.S. Calcium Hydroxide in the Treatment of Pulpless Teeth with Associated Pathology, J. Brit Endo Society, 8(2): 74-93 July 1975)

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