Instructions for Use ASBAGEN®

ANALLERGIC, STERILE, REABSORBABLE HAEMOSTATIC PADS OF HORSE COLLAGEN

PRODUCT DESCRIPTION:

Haemostatic ASBAGEN® is composed of purified horse collagen. The product in the form of lyophilized pads, is sterile, anallergic, and reabsorbable. Its haemostatic activity is a specific property of collagen, so that when collagen comes into contact with blood there is an aggregation of the platelets (thrombocytes) which activate the factors of coagulation and these together with the plasmatic factors, lead to the formation of fibrin and therefore of the clot. Due to enzymatic degradation, ASBAGEN® is completely reabsorbed in about three weeks.

INDICATIONS:

ASBAGEN® is suggested in dentistry as adjunctant in the control of capillary hemorrhages. The size 1x1 x 0,5 cm is suggested in the alveolar surgery and in dental extractions. ASBAGEN® is subsequently indicated in patients affected by haemocoagulation deficiency.

CONTRAINDICATIONS:

ASBAGEN® should not be applied on bone surfaces where methyl-methacrylate-adhesive is to be used, as collagen may reduce the bonding strength of methacrylate.

SPECIAL PRECAUTION IN USE:

ASBAGEN® is damaged by hydrolysis after sterilization is effected by autoclaving, consequently, when the product pack is opened and unused, the contents are rendered non-sterile and should be discarded. The gamma ray sterility of the product, unless the packing is opened or damaged, is assured. The pads should not be used on infected wounds.

APPLICATION:

ASBAGEN® is applied directly to the bleeding surface with light pressure. The pad can be cut to the size required. The amount of product necessary and the pressure time required depends on the type and area of bleeding to be controlled.

Haemostasis depends on the type of surgery and degree of pre-treatment: it usually occurs between 2 and 4 minutes. ASBAGEN® does not disperse when wet.

PACKAGING:

ASBAGEN® is supplied as lyophilized pads in the following size: boxes containing 60 pads 1 x 1 x 0.5 cm in blister pack

Restricted to use by dental professional only.

Dispose of unused product in accordance with the official rules and regulations in force.

HANDLING AND STORAGE:

Store packing in a cool, dry place.

☞ Shelf life is indicated on outer package

🏢 **MOLTENI DENTAL S.r.l. Via Ilio Barontini, 8 Loc. Granatieri - 50018 Scandicci FI**

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ASBA is a registered trademark of LA MAISON DENTAIRE SA.